



EMODIAL®

Via Finati, 4-4/A - Ferrara - ITALY
Tel. +39 0532 978198 - Fax +39 0532 978414
www.emodial.com - info@emodial.com

Ref. GB0020 - Rev. 11/2021



EMODIAL®

EXIT-PRO® AG
SoftTouch

Catheter dressing



DESCRIPTION

EXIT-PRO[®] AG SoftTouch is an absorbent non-adherent dressing with silver ion release on a non-woven fabric (NW) support with "SoftTouch" adhesive technology.

INTENDED USE

- To be used as a dressing for the insertion points of vascular and non-vascular percutaneous devices and as a mechanical barrier for the compression and absorption of exudates.
- Does not require secondary dressings.
- Compatible with all types of standard use catheter.

PURPOSE OF USE

The dressing performs a complete protective action:

- **the silver ions releasing film:** guarantees immediate and long-lasting antibacterial efficacy, up to 7 days, helping to reduce the risk of exit-site infection associated with the use of percutaneous devices;
- **the support with "SoftTouch" adhesive technology:** protects the skin from mechanical damage caused by the continuous removal of patches (as a consequence of repeated treatments), favoring tissue repair and acting as a treatment and prevention of MARSI;
- prevents the onset of redness and irritation due to prolonged application of patches;
- optimal solution for patients with a tendency to dry or scaly skin, typical of patients undergoing, for example, dialysis or oncological treatments.




The delicacy of the SoftTouch adhesive combines with the antibacterial and preventive action of silver.

Ideal for chronic patients or for patients with delicate or sensitized skin.



Traditional adhesive or SoftTouch?

Traditional adhesives

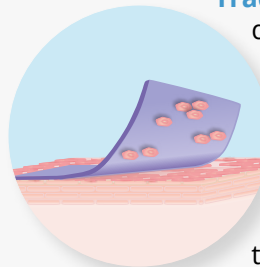


do not adhere uniformly to the skin, but leave gaps on the surface which are subsequently filled.

A traction is exerted on the skin which makes the fixing stronger, suitable for non-repeated treatments.

In SoftTouch adhesives,

adhesion to the skin is uniform and has a low surface tension.

Removal is less traumatic making this type of adhesive ideal for repeated treatments or for delicate and sensitive skin.

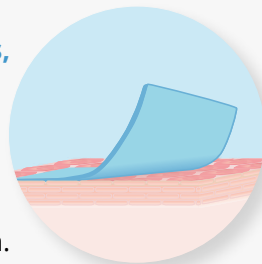

INSTRUCTIONS FOR USE

1. Disinfect the exit site and let it dry completely.
2. Raise the two edges of the protective paper of the patch.
3. Place the pad of **EXIT-PRO[®] AG SoftTouch** in contact with the skin at the exit-site, making the end of the percutaneous device come out of the prepared cut, and fix it by gently pressing the surrounding adhesive.
4. Replace **EXIT-PRO[®] AG SoftTouch** at every session or every time it gets wet, dirty or it is necessary to inspect the exit-site.

REFERENCES

| CODE | PLASTER (cm) | PAD (cm) | HOLE ø (cm) | PCS/ PACK | PACKS/ BOX |
|----------|--------------|----------|-------------|-----------|------------|
| S0810/AG | 10 x 8 | 5 x 4 | 0.4 | 1 | 25 |