



**EMODIAL**



**EMODIAL**

Via Finati, 4/A - Ferrara - ITALY

Tel. +39 0532 978198 - Fax +39 0532 978414

[www.emodial.com](http://www.emodial.com) - [info@emodial.com](mailto:info@emodial.com)

**CE**  
0373

Ref. GB004 - Rev. 07/2019

**SOVAN<sup>®</sup>**

Fistula Dressing



### DESCRIPTION

**SOVAN** is a highly absorbing two layers hemostatic pad set on hypoallergenic, air permeable adhesive. Also suitable for patients who require intravenous treatment and blood transfusion.

### INTENDED USE

Compressive hemostatic action, specific for post dialysis arteriovenous access care. To be used for the hemostasis after the removal of the fistula needles.

### PURPOSE

Designed to stop bleeding and to reduce the hemorrhagic risk, thanks to the compression on the needle holes, also working as a barrier against the risk of infections.

The non-adherent, biocompatible layer on the pad doesn't stick to the skin **avoiding the clot removal and the risk of re-bleeding.**

The adhesive helps to keep the pad in position.



### DIRECTIONS FOR USE



**1**  
Remove protective paper and fix the first half of the dressing to the skin.



**2**  
Put the pad over the needle site, press and remove needle.



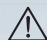

**3**  
Remove the second half of the paper and fix tightly the other half of the dressing to the skin. Hold in place until hemostasis is achieved.

### REFERENCES

Data and technical specifications can be changed without notice.

CODE	PLASTER (cm)	PAD (cm)		ACTIVE SURFACE PLASTER AREA	PCS / PACK	PACKS / BOX
		Size ø	Thickness			
S0074	15 x 5	2.6	0.6 ± 1	75 cm <sup>2</sup>	1	500
S0075	15 x 5	2.6	0.6 ± 1	75 cm <sup>2</sup>	2	500
S0079	10 x 5	2.6	0.6 ± 1	50 cm <sup>2</sup>	2	500
S0079-1	10 x 5	2.6	0.6 ± 1	50 cm <sup>2</sup>	1	700

CLASSIFICATION • CLASS I MEDICAL DEVICE  
 BIOCOMPATIBLE • ACCORDING TO UNI EN ISO 10993 STANDARDS  
 STERILIZATION • GAMMA RAYS  
 EXPIRATION • 5 YEARS

  • SINGLE-USE